Désignation : Notice SOLO CARE AQUA

Code Article : 6157/05-1

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Plan :

Date : 27-02-2021 12:39

Épreuve 3 page 1

Proc. : BLACK

DECOUPE

Date/Validation Client :

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EN IMPORTANT PRODUCT USE AND SAFETY INFORMATION, PLEASE READ CAREFULLY AND RETAIN FOR FUTURE REFERENCE. INCORRECT CONTACT LENS CARE CAN CAUSE SERIOUS PROBLEMS WITH THE EYES.

SOLOCARE AQUA® is a one bottle multi-purpose/disinfecting solution that cleans, rinses, disinfects, stores, removes protein and lubricates all soft contact lenses. Isotonic, aqueous anti-microbial/anti-bacterial solution containing: polyhexanide 0.0001% (active ingredient and preservative), HydroLock® (dexpanthenol and sorbitol), sodium phosphate, tromethamine, polyoxamer 407, disodium edetate.

⁽¹⁾⁽²⁾⁽³⁾⁽⁴⁾⁽⁵⁾ Processed by aseptic techniques. The MicroBlock® antibacterial lens case contains silver.

*active against eye-pathogen germs listed in ISO 14729

INSTRUCTIONS FOR USE

1. Wash your hands and dry them with a clean towel
2. Open the lens case
3. Remove lens and place in the palm of your hand
4. Apply at least 3 drops of SOLOCARE AQUA® to each lens surface (see picture 1)
5. Gently rub the lens using a circular motion for at least 10 seconds
6. Rinse the lens thoroughly with SOLOCARE AQUA® while rubbing the lens gently between the thumb and forefinger for at least 10 seconds (see picture 2)
7. Place the lens in the lens case and fill one side to the line with fresh SOLOCARE AQUA®
8. Replace cap and screw closed
9. Repeat procedure with your other lens
10. ALLOW YOUR LENSES TO SOAK FOR AT LEAST 5 MINUTES

11. Your contact lenses may now be inserted directly into your eyes
12. Discard solution from lens case, rinse it with fresh SOLOCARE AQUA® and dose until next use
13. If the lenses have been stored in SOLOCARE AQUA® for more than 30 days before wearing, you must discard the solution from the case and repeat the manual cleaning process and disinfect the lenses in SOLOCARE AQUA® for at least 5 minutes before wearing them

KONTRAINDICATIONS (reasons not to use)

- Do not use if you are sensitive to any of the ingredients or if you are allergic to silver or other metals
- Never use SOLOCARE AQUA® for heat disinfection

PRECAUTIONS

- Do not use after expiry date
- Not to initially use the product, if the safety ring is damaged.
- Close bottle after use
- NEVER transfer SOLOCARE AQUA® to another container
- NEVER dilute or mix SOLOCARE AQUA® with any solution as this will destroy its performance
- NEVER ALLOW CONTACT LENSES TO COME INTO CONTACT WITH NON-STERILE LIQUIDS (INCLUDING TAP WATER AND SALIVA)
- AS MICROBIAL CONTAMINATION CAN OCCUR, WHICH MAY LEAD TO PERMANENT EYE DAMAGE
- ALWAYS USE THE SOLOCARE AQUA® SYSTEM COMPONENTS INCLUDING THE LENS CASE
- KEEP OUT OF REACH OF CHILDREN
- Consult your eye care professional before changing solutions
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

WARNINGS

- Always use fresh SOLOCARE AQUA® solution – never re-use solution in your lens case for a second time as efficacy is no longer provided
- NEVER transfer SOLOCARE AQUA® to another container
- NEVER dilute or mix SOLOCARE AQUA® with any solution as this will destroy its performance
- NEVER ALLOW CONTACT LENSES TO COME INTO CONTACT WITH NON-STERILE LIQUIDS (INCLUDING TAP WATER AND SALIVA)
- AS MICROBIAL CONTAMINATION CAN OCCUR, WHICH MAY LEAD TO PERMANENT EYE DAMAGE
- ALWAYS USE THE SOLOCARE AQUA® SYSTEM COMPONENTS INCLUDING THE LENS CASE
- KEEP OUT OF REACH OF CHILDREN
- Consult your doctor or eye care professional before using any medication with contact lenses

ADVERSE REACTIONS

If you experience persistent discomfort or irritation remove your lenses, stop using the product and consult your eye care professional.

MicroBlock® LENS CASE

The MicroBlock® antibacterial lens case may slightly discolour over time; this discolouration does not affect the anti-microbial properties of the lens case.

SOLOCARE AQUA® is available in 90 ml and 360 ml bottles, including a disposable lens case.

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